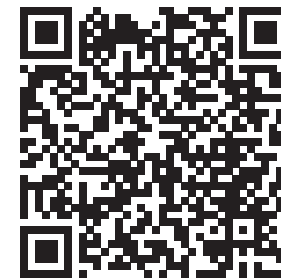
Complete Video Guide
Scan the QR code to access detailed videos
on using the scalp cooling cap.

EN

**Instruction for Use
Hot Cold Pack (cooling cap)**

EN Scalp Cooling Cap Guide

Thank you for choosing our cooling cap. This device is designed to help reduce hair loss during chemotherapy treatment. We know this process can feel challenging, so we want everything related to the use of the cap to be as simple as possible.

1 Box Reception and Initial Preparation

Step 1: Upon receiving the box, open it and remove the caps from their individual packages.

Step 2: Place the caps, in their plastic bags, in the freezer for at least 24 hours to reach the optimal temperature of -20°C.

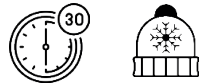
2 Preparing the Insulated Bag and Placing the First Cap

Step 3: Take the caps out of the freezer as close as possible to the start time of your treatment, and place them in the insulated bag to keep them at the right temperature until you're ready to use them.

Step 4: Put on the first cap 30 to 45 minutes before starting your chemotherapy session.

Note: If the cap is too stiff when you first take it out of the freezer, let it sit for a few minutes to make it easier to put on. It's normal for the cap to feel rigid at first, but it will gradually become more flexible.

Step 5: Adjust the cap to ensure it fully covers your scalp without leaving gaps. Make sure it is positioned correctly for effective cooling.

3 During Chemotherapy

Step 6: Every 30 minutes, replace the cap with a new, cold one from the insulated bag. This will keep your scalp at the correct temperature.

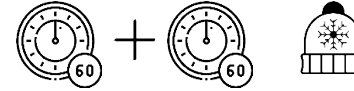
Step 7: Change caps smoothly and at a steady pace, but try to be efficient to maintain the cooling effect.

Note: Make sure you have enough caps for the entire duration of the treatment.

4 After Chemotherapy

Step 8: After the chemotherapy session, put on a new cap and wear it for an additional 60 to 120 minutes.

Step 9: If necessary, change the cap every 30 minutes to maintain the cooling effect during this post-treatment period.

+ Final Tips and Precautions

- **Plan Ahead:** Ensure you have enough frozen caps ready for each chemotherapy session. Set a timer to remind yourself when to change caps.
- **Stay Warm:** To help ease the sensation of cold during treatment, consider wearing a scarf, gloves, or blanket.
- **Comfort:** If you feel uncomfortable or the cap feels too tight, adjust it carefully without losing contact with your scalp.
- **Efficient Cap Changes:** Change the caps smoothly but efficiently to maintain the cooling effect.
- **Do Not Interrupt Cooling:** Avoid leaving gaps in time without a cap during treatment to achieve the best results.

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LOT 20250501

Manufactured in China

For any concerns relating to this product please contact the manufacturer listed below with LOT and UDI code listed above.

UKRP Anhui Kamt Ice Pack Co. Ltd
No. 2 Xiangyang Road, Sanshipu Town, Jin'an District, Lu'an City, Anhui Province

UKRP SUNGO Certification Company Limited
3rd floor, 70 Gracechurch Street, London, EC3V 0HR

EC REP SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7, 2909 VA Capelle aan den IJssel, The Netherlands

UKRP **Title/Meaning:** Manufacturer
Function/description: To identify the manufacturer of a product.

UKRP **Title/Meaning:** Date of manufacture
Function/description: To indicate the date on which a product was manufactured.

REF **Title/Meaning:** Catalogue number
Function/description: To identify the manufacturer's catalogue number.

LOT **Title/Meaning:** Batch code
Function/description: To identify the manufacturer's batch or lot code.

CE Mark of conformity according to the European MDR (EU) 2017/745

UK CA Mark of conformity according to the UK MDR2002

UKRP **Title/Meaning:** Operator's manual or Operating instructions
Function/description: To identify the location where the operator's manual is stored or to identify information that relates to the operating instructions. To indicate that the operating instructions should be considered when operating the device or control close to where the symbol is placed.

UKRP **Title/Meaning:** Do not use if package is damaged
Function/description: To indicate that the device must not be used if the package holding the device is damaged, for example on packaging of medical devices.

UKRP **Title/Meaning:** Keep away from sunlight
Function/description: To indicate that transport package shall not be exposed to sunlight.

UKRP **Title/Meaning:** Keep away from rain
Function/description: To indicate that the transport package shall be kept away from rain and in dry conditions.

UKRP **Title/Meaning:** Temperature limit
Function/description: To indicate the maximum and minimum temperature limits at which the item shall be stored, transported or used.

EC REP **Title/Meaning:** Authorized representative in the European Community
Function/description: Indicates the Authorized representative in the European Community.

UKRP **Title/Meaning:** Authorized representative in the UK
Function/description: Indicates the Authorized representative in the UK.

MD **Name:** Medical Device
Description: Indicates the item is a medical device.

UDI **Name:** Unique device identifier
Description: Indicates a carrier that contains unique device identifier information.

UKRP Indicates the date after which the medical device is not to be used.

UKRP Symbol for "CAUTION"
This symbol is essentially a safety symbol and should be used to highlight the fact that there are specific warnings or precautions associated with the device, which are not otherwise found on the label. The symbol "Caution" is still sometimes used to have the meaning of "Attention, see instructions for use".

UKRP Symbol for "Importer"
Indicates the entity importing the medical device into the locale.

UKRP Symbol for "Distributor"
Indicates the entity distributing the medical device into the locale.

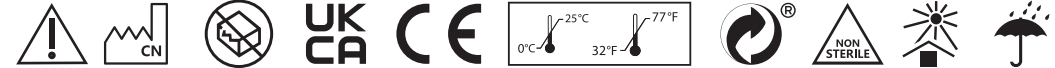
UKRP Symbol for "Country of manufacture"
To identify the country of manufacture of products.

UKRP Valid only in Spain Mandatory signs for garbage recycling are used to indicate that products and packaging should be sorted and recycled or sent to the recycling point.

UKRP Symbol for "Non sterile product"
Indicates a medical device that has not been subjected to a sterilization process. This symbol should only be used to distinguish between identical or similar medical devices sold in both sterile and non-sterile conditions.

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